




# QUICK START GUIDE



## 6.2 ColostrumFIT Checklist Prepare colostrum bank


**1** Start of calving

**2** ColostrumFIT is ready on Standby

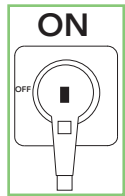


**3** Calf is born

**4** Milk cow as soon as possible

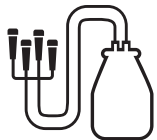


**5** Check the colostrum quality  
(Colostrometer in the green range;  
Refractometer >22 Brix%)

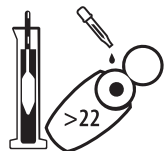


**6** Fill colostrum in ColostrumBAG

**7** Use Profile 1 to pasteurize colostrum  
(Duration: about 2 hours)

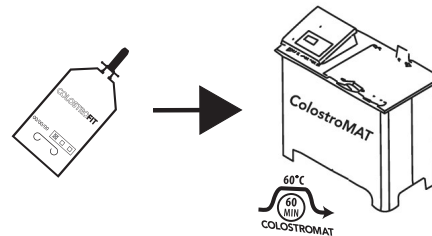


**8** Feed calf as soon as possible (within 4 hours) with ColostrumBAG

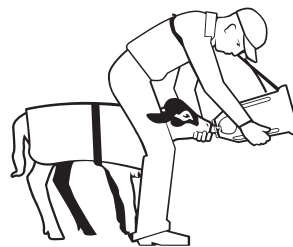

**9** Dispose ColostrumBAG



**10** Pasteurize remaining colostrum with Profile 2

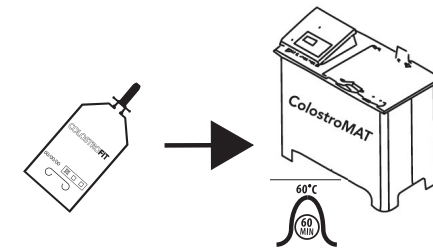

**11** Freeze and store pasteurized colostrum

